Door transmitter

BE1411

Návod k použití	CZ
Benutzerhandbuch	DE
Brugervejledning	DK
User manual	EN
Manual del usuario	ES
Käyttöopas	FI
Mode d'emploi	FR
Εγχειρίδιο χρήσης	GR
Használati útasítás	HU
Manuale utente	IT
Gebruiksaanwijzing	NL
Brukerhåndbok	NO
Instrukcja obsługi	PL
Manual do utilizador	PT
Bruksanvisning	SE
Návod na použitie	SK
Installer's guide	EN

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
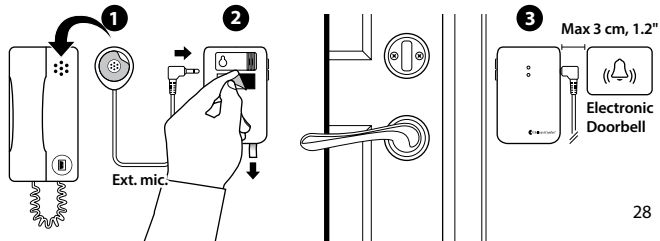
- 1 Remove the protective film from the external microphone (sold separately) and attach it to the intercom's speaker. Connect it to the transmitter.
- 2 Pull the battery tab to start the transmitter. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 3 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. **Note:** The intercom and doorbell must be at least 25 cm, 10" apart to avoid sound interference.

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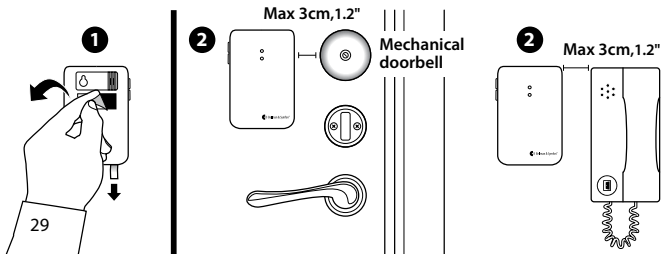
- 1 Pull the battery tab to start each door transmitter. Clean the wall surface with the wet wipe and remove the protective film from the Velcro.
- 2 Mount one of the door transmitters to the left of the doorbell and the other to the left of the intercom's speaker, as close to the sound sources as possible.

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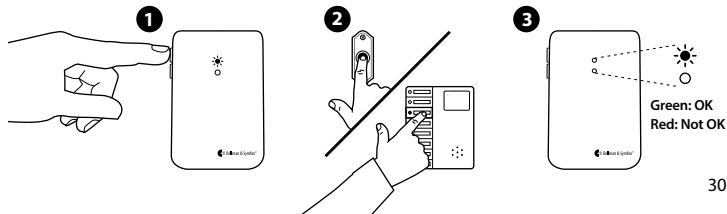
- 1 Press and hold button 1 until the top LED start to blink. Release it to start the recording.
- 2 **For door buzzers:** Press and hold the doorbell for at least 8 seconds.
For door chimes: Press the doorbell at least 8 times.
- 3 Press button 1 on the transmitter to stop the recording. If the LED lights up in red, you need to repeat steps 1 and 2. Following the same steps, you can use button 2 to record the sound of your intercom.

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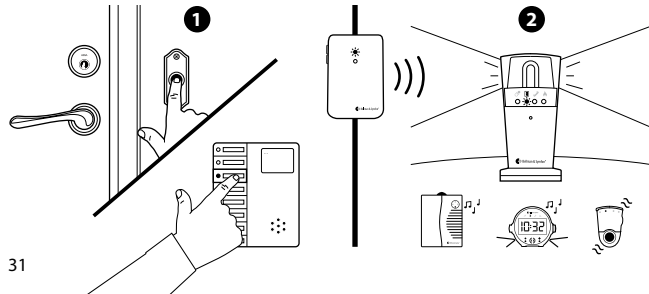Technical information

Batteries	2 x 1.5 V AA (LR6) lithium or alkaline batteries
Power consumption	Active < 70 mA, Idle position < 15 μ A
Operating time	Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via the test buttons, the internal or external microphone, via electromagnetic detection or the external trigger input
Radio frequency	314.91 MHz, 433.92 MHz and 868.30 MHz depending on region
Max output power	4.7 dBm
Coverage	50 – 250 m (55 – 273 yd) depending on radio frequency. Coverage is reduced by walls, large objects and other radio transmitters
Number of radio keys	64 as standard. Up to 256 with optional software
Accessories	BE9199 External microphone 2.5 m (2.7 yd), BE9200 External microphone 0.75 m (0.8 yd)

Additional information

Operation, transport and storage	For information, see the Installer's Guide .
Maintenance and cleaning	Maintenance free. Clean with a dry cloth. Don't use household cleaners, aerosol sprays, solvents, alcohol, ammonia, or abrasives.
Safety and regulations	For information, see the Installer's Guide .
Service and support	For information, see the Installer's Guide .

Troubleshooting

If	Try this
The LEDs blink in orange every minute	<ul style="list-style-type: none">• Change the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.
The LEDs blink in orange every second	<ul style="list-style-type: none">• There are other competing sound sources around the door transmitter. Switch them off or turn them down.• Disconnect the External microphone to make sure it is not faulty.
The green LED lights up when I press the doorbell - but the receiver is not activated	<ul style="list-style-type: none">• Check the transmitter batteries and the receiver batteries and connections.• Move the receiver closer to the door transmitter.• Check that the door transmitter and the receiver are set to the same radio key. For more information, see Changing the radio key in the Installer's Guide.

If	Try this
The transmitter's green LED doesn't light up when I ring the doorbell	<ul style="list-style-type: none">• Make sure you have mounted the transmitter to the left of the doorbell sound source, no more than 3 cm apart.• If you are using an external microphone, make sure its connected. If you are using an intercom, make sure that the distance between the intercom and transmitter is at least 25 cm or that the internal microphone is turned off.• Try again to teach the transmitter to recognize the sound of your doorbell, see Optimizing the doorbell detection.• If the doorbell signal varies a lot, change to magnetic detection, see Electromagnetic detection in the Installer's Guide.
The receiver is activated for no reason	<ul style="list-style-type: none">• Another Visit system nearby might trigger your system. Change the radio key on all units, see Changing the radio key in the Installer's Guide.

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Introduction

Function

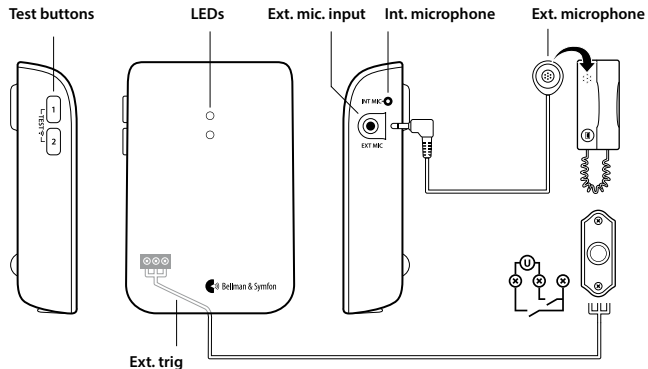
This door transmitter is part of the Visit smart home system that helps people with hearing loss to keep tabs on their home. The door transmitter can be programmed to identify virtually any doorbell sound.

Installation

Depending on the doorbell configuration, choose one of the below procedures:

- **Single sound source, using the internal microphone** p.135
- **Single sound source, using an external microphone** p.136
- **Intercom + electronic doorbell** p.137
- **Intercom + mechanical doorbell** p.138
- **Hardwiring the doorbell** p.139
- **Using electromagnetic detection** p.142


Overview

Basic installation – single sound source

Using the transmitter's internal microphone

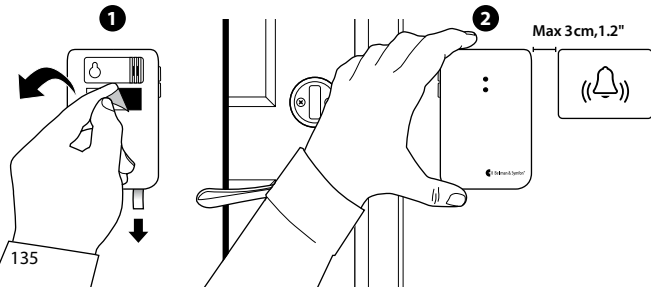
- 1 Pull the battery tab to start the transmitter. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 2 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. You can also use the supplied screw and plug.

Alternative installation – single sound source


Using the external microphone accessory

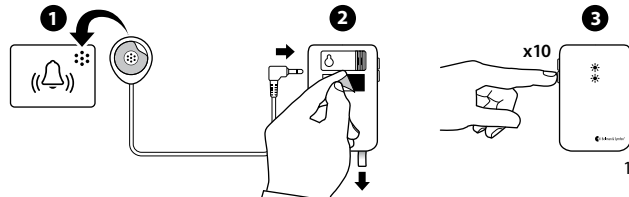- 1 Connect the external microphone (sold separately) to the transmitter. Remove the protective film and attach it to the intercom's speaker.
 - 2 Pull the battery tab to start the transmitter. Remove the protective film from the Velcro and mount the transmitter on the wall.
 - 3 Within 2 min, press the lower button 10 times to turn off the internal microphone. The LEDs will blink 3 times in red to confirm.
- Note:** If you exceed 2 minutes, you need to restart the transmitter by removing the batteries and putting them back in again.

Installation – intercom and electronic doorbell

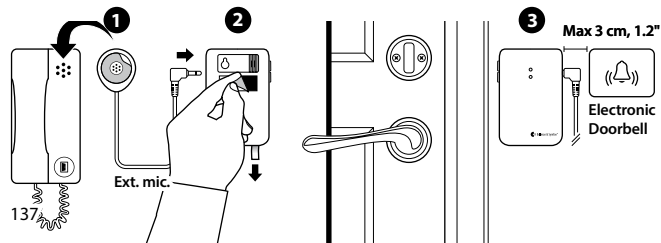
- 1 Remove the protective film from the external microphone (sold separately) and attach it to the intercom's speaker. Connect it to the door transmitter.
- 2 Pull the battery tab to start the transmitter. Clean the wall with the wet wipe and remove the protective film from the Velcro.
- 3 Mount the door transmitter to the left of the doorbell's sound source, as close as possible. **Note:** The intercom and doorbell must be at least 25 cm, 10" apart to avoid sound interference.

Installation – intercom and mechanical doorbell



If you have an intercom and a mechanical doorbell, you may need two door transmitters, i.e. one for each sound source.

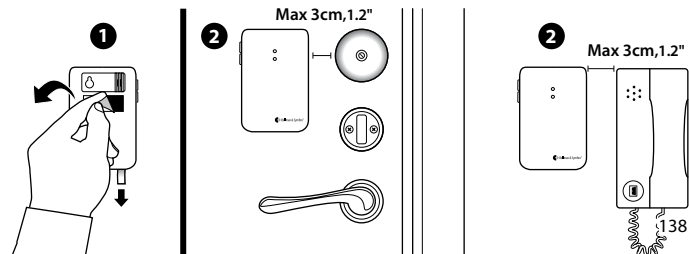- 1 Pull the battery tab to start each door transmitter. Clean the wall surfaces with the wet wipe and remove the protective films from the Velcros.
- 2 Mount one of the door transmitters to the left of the doorbell and the other to the left of the intercom's speaker, as close to the sound sources as possible.





Installation – hardwiring your doorbell

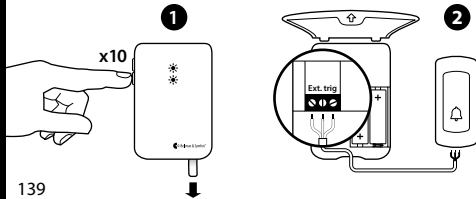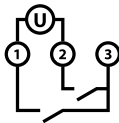- 1 Pull the battery tab to start the transmitter. Within 2 min, press the lower button 10 times to turn off the internal mic. The LEDs will blink 3 times to confirm.
- 2 Remove the front cover to access the screw terminal. There are 3 connection points:
 - Use 1 and 2 to connect an **active** switch, like a relay with current (2–30VDC, polarity independent or 2–24VAC, 5 - 150Hz).
 - Use 2 and 3 to connect a **passive** switch, like a push button.
- 3 Put the front cover back, remove the protective film from the Velcro and mount the door transmitter on the wall.

139

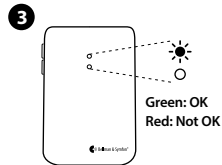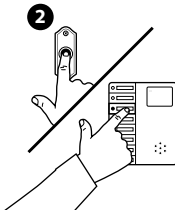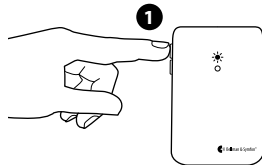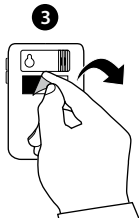140

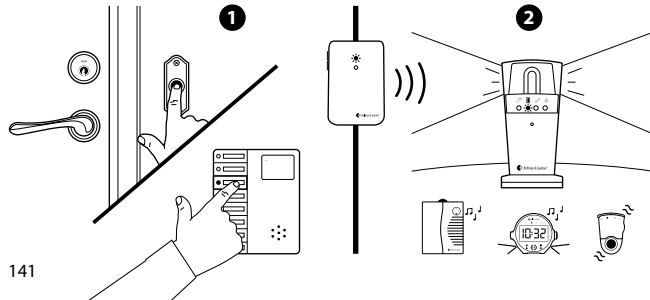
Optimizing the doorbell detection

The next step is to teach the transmitter to recognize your doorbell.

- 1 Press and hold button 1 until the top LED start to blink. Release it to start the recording.
- 2 **For door buzzers:** Press and hold the doorbell for at least 8 seconds.
For door chimes: Press the doorbell at least 8 times.
- 3 Press button 1 on the transmitter to stop the recording. If the LED lights up in red, you need to repeat steps 1 and 2. If you have an intercom, use button 2 and follow the steps above to record its sound.

Testing

- 1 Ring the doorbell or intercom.
- 2 The LED on the Visit receiver lights up in green and the receiver starts to sound, flash or vibrate (depending on the model). If it doesn't react, see **Troubleshooting**.

Using electromagnetic detection

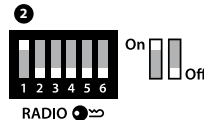
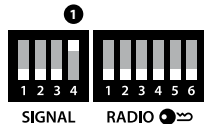
The door transmitter can be set to detect electromagnetic fields emitted by electromechanical doorbells. Here is how it's done:

- **Activating electromagnetic detection**
Move the 4th signal-switch ① to the Up=On position.
- **Deactivating electromagnetic detection**
Move the 4th signal-switch to the Down=Off position.

Changing the radio key

If you install multiple systems, you can set a unique radio key for each system to avoid radio interference. Move any radio key switch ② to the up=on position to change the radio key.

Note: All Visit products must be set to the same radio key in order to operate as a group.

Default signal pattern

When the door transmitter is activated by a microphone or a connected doorbell, the following happens:

- The transmitter's LED lights up to show that it's signalling the receiver.
- The receiver's LED lights up and it starts to sound, flash or vibrate with a certain pace, called signal pattern.

The transmitter and the connected accessories determine the signal pattern and the default is as follows:

Transmitter		Receiver signal pattern			
Source	LED	LED	Sound	Vibration	Flash
Internal microphone	Green, top	Green light	1 x door chime, low	Slow ■□□□	Yes
External microphone	Green, bottom	Green blinks	2 x door chime, high	Slow ■□□□	Yes
Connected doorbell	Green, top	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes

Changing the signal pattern

The transmitter controls the signal pattern. Open the transmitter front cover and move the signal switches according to the tables on page 145–148 to change it.

Note: Switch no. 4 is for electromagnetic detection. The signal pattern may differ depending on the receivers.

Deleting the recorded sounds

To delete all recorded sounds, press and hold button 1 and 2 simultaneously for 5 s. **Note:** All radio key settings and signal settings are kept intact.

Turning the internal microphone back on

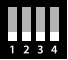
Restart the transmitter by removing the batteries and putting them back in. Within 2 minutes, press the lower button 10 times to turn the internal microphone back on. The LEDs will blink 3 times in green to confirm.

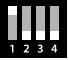
Signal output for 868 MHz and 315 MHz

Transmitter		Receiver signal pattern			
Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green blink	2 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	2 x green blinks	2 x door chime low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	3 x orange blinks	Baby melody	Fast ■■■■■■	Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	Orange blinks	Baby melody	Fast ■■■■■■	Yes

Transmitter		Receiver signal pattern			
Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Int. mic. / learned signal 1 / test	Green light	1 x door chime low	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Connected doorbell	Yellow blinks	2 x ring signal, high	Medium ■□■□	Yes
	Int. mic. / learned signal 1 / test	3 x green blinks	1 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	Red + orange light	Emergency siren	Long ■■□□	Yes
	Int. mic. / learned signal 1 / test	Green blinks	2 x door chime, high	Slow ■□□□	Yes
	Ext. mic. / learned signal 2	2 x green blinks	2 x door chime, low	Slow ■□□□	Yes
	Connected doorbell	Red light	Fire horn	Long ■■□□	Yes

Signal output for 433 MHz

Transmitter		Receiver signal pattern			
Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Long 	Yes
	Ext. mic. / learned signal 2	Green light	1 x ring signal, high	Medium 	Yes
	Connected doorbell	Green light	1 x door chime, low	Long 	Yes
	Int. mic. / learned signal 1 / test	Green light	1 x ring signal, high	Medium 	Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Long 	Yes
	Connected doorbell	Green light	1 x ring signal, high	Medium 	Yes
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Long 	Yes
	Ext. mic. / learned signal 2	Green light	1 x ring signal, high	Medium 	Yes
	Connected doorbell	Orange light	Baby melody	Long 	Yes
	Int. mic. / learned signal 1 / test	Green light	1 x ring signal, high	Medium 	Yes
	Ext. mic. / learned signal 2	Green light	1 x door chime, low	Long 	Yes
	Connected doorbell	Orange light	Baby melody	Long 	Yes

Transmitter		Receiver signal pattern			
Switch	Source	LED	Sound	Vibration	Flash
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Long 	Yes
	Ext. mic. / learned signal 2	Green light	1 x ring signal, high	Medium 	Yes
	Connected doorbell	Yellow light	1 x ring signal, low	Medium 	Yes
	Int. mic. / learned signal 1 / test	Green light	1x door chime, low	Long 	Yes
	Ext. mic. / learned signal 2	Green light	1x ring signal, high	Medium 	Yes
	Connected doorbell	Orange light	Baby melody	Long 	Yes
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Long 	Yes
	Ext. mic. / learned signal 2	Green light	1 x ring signal, high	Medium 	Yes
	Connected doorbell	Red + orange light	Emergency siren	Continuous 	Yes
	Int. mic. / learned signal 1 / test	Green light	1 x door chime, low	Long 	Yes
	Ext. mic. / learned signal 2	Green light	1 x ring signal, high	Medium 	Yes
	Connected doorbell	Red light	Fire horn	Continuous 	Yes

Troubleshooting

If	Try this
The LEDs blink in orange every minute	<ul style="list-style-type: none">• Change the batteries. Only use 1.5 V AA (LR6) lithium or alkaline batteries.
The LEDs blink in orange every second	<ul style="list-style-type: none">• There are other competing sound sources around the door transmitter. Switch them off or turn them down.• Disconnect the External microphone to make sure it is not faulty.
The green LED lights up when I press the doorbell - but the receiver is not activated	<ul style="list-style-type: none">• Check the transmitter batteries and the receiver batteries and connections.• Move the receiver closer to the door transmitter.• Check that the door transmitter and the receiver are set to the same radio key. For more information, see Changing the radio key.

If	Try this
The transmitter's green LED doesn't light up when I ring the doorbell	<ul style="list-style-type: none">• Make sure you have mounted the transmitter to the left of the doorbell sound source, no more than 3 cm apart.• If you are using an external microphone, make sure its connected. If you are using an intercom, make sure that the distance between the intercom and transmitter is at least 25 cm or that the internal microphone is turned off.• Try again to teach the transmitter to recognize the sound of your doorbell, see Optimizing the doorbell detection.• If the doorbell signal varies a lot, change to magnetic detection, see Electromagnetic detection.
The receiver is activated for no reason	<ul style="list-style-type: none">• Another Visit system nearby might trigger your system. Change the radio key on all units, see Changing the radio key.

Technical information

Batteries	2 x 1.5 V AA (LR6) lithium or alkaline batteries.
Power consumption	Active < 70 mA, Idle position < 15 µA
Operating time	Alkaline ~ 5 years, Lithium ~ 10 years
Activation	Via the test buttons, the internal or external microphone, via electromagnetic detection or the external trigger input
Radio frequency Coverage	314.91 MHz, 433.92 MHz and 868.30 MHz depending on region 50 – 250m (55 - 273yd) depending on radio frequency. Coverage is reduced by walls, large objects and other radio transmitters.
Number of radio keys	64 as standard. Up to 256 with optional software.
Accessories	BE9199 External microphone 2.5m (2.7yd), BE9200 External microphone 0.75m (0.8yd).

Additional information

Operating temperature	0° – 35° C, 32° – 95° F
Transport and storage temp.	-10° – 50° C
Relative humidity	15% – 90% (non condensing)
Maintenance and cleaning	Maintenance free. Clean with a dry cloth. Don't use household cleaners, aerosol sprays, solvents, alcohol, ammonia, or abrasives.
Warranty and service	If the product appears to be damaged or doesn't function properly, follow the advice in this booklet. If it still doesn't function properly, contact your retailer for info about obtaining warranty service.

Directive 2014/53/EU. Operating Frequency Range: 868.3MHz. Max Output Power: 4.7dBm.

Hereby, Bellman & Symfon declares that this Door transmitter is in compliance with the essential requirements and other relevant provisions of directives: RED

Use and disposal of batteries

Replace only with the same or equivalent type recommended by the manufacturer. Please dispose of old, defective batteries in an environmentally friendly manner in accordance with the relevant legislation in your country.

FCC compliance statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment gener-

ates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an exper-

rienced radio/TV technician for help.

For a complete Declaration of Conformity please contact the Bellman & Symfon European office.

Industry Canada statement

This device complies with Industry Canada Licence-exempt RSS-210. Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must

European office

Bellman & Symfon Europe AB
Södra Långebergsgatan 30
436 32 Askim
Sweden
Phone: +46 31 68 28 20
E-mail: info@bellman.com

accept any interference, including interference that may cause undesired operation of the device.

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This Class B digital apparatus complies with the Canadian ICES-003.

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Bellman & Symfon Asia Ltd.